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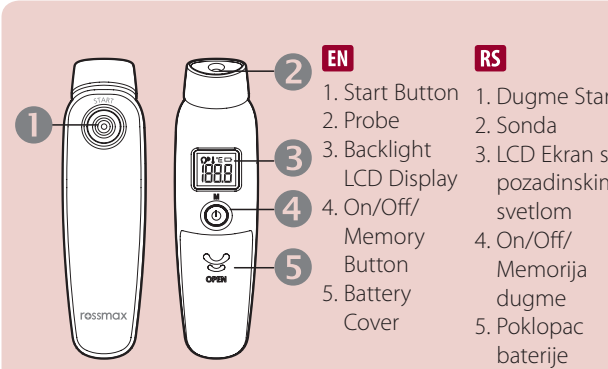
Electromagnetic Compatibility Information This device needs to be installed and put into service in accordance with the information provided in the user manual. 2. WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the HA500, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result. If higher IMMUNITY TEST LEVELS than those specified in Table 9 are used, the minimum separation distance may be lowered. Lower minimum separation distances shall be calculated using the equation specified in 8.10.			
Manufacturer's declaration-electromagnetic immunity The HA500 is intended for use in the electromagnetic environment specified below. The customer or the user of the HA500 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms: 0.15 MHz – 80 MHz 6 Vrms: in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80 µV AM at 1 kHz	Not applicable Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of the HA500 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: d = 1.2 √P; d = 1.2 √P 800MHz to 800 MHz; d = 2.3 √P 800MHz to 2.7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Interference may occur in the vicinity of equipment marked with the following symbol:⚡
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2.7 GHz 80 µV AM at 1 kHz	10 V/m 80 MHz – 2.7 GHz 80 µV AM at 1 kHz	
NOTE1: At 80 MHz and 800 MHz, the higher frequency range applies. NOTE2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. More information on EMC compliance of the device can be obtained from Rossmax website: www.rossmax.com			

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EN English

Introduction

This Rossmax non-contact temple thermometer is intended for measuring human body temperature; while is not intended as a substitute for consultation with your physician. This thermometer has been clinically tested in the hospital based on the regulatory standard protocol and proven to be safe and accurate when used in accordance to the operating instruction manual.

- The quality of the infrared temple thermometer has been verified and conforms to the provisions of the EC council directive 93/42/EEC (Medical Device Directive) Annex I essential requirements and applied harmonized standards. Comply with ASTM E1965-98(2016).

- This thermometer converts the temple temperature to display its "oral equivalent."(according to the result of the clinical evaluation)

Please read the instruction manual carefully before using the product and be sure to keep this manual.

Function

Temple Temperature	The thermometer has been designed for practical use. It's not meant to replace a visit to the doctor. Please also remember to compare the measurement result to your regular body temperature.
Object Mode	The object mode shows the actual, unadjusted surface temperatures, which is different from the body temperature. It can help you to measure any object other than body surface, for example the baby's milk.
Fever Alarm	If thermometer detects a body temperature $\geq 37.5^{\circ}\text{C}(99.5^{\circ}\text{F})$ there will be four beeps sound with flashing reading to warn the user for potential fever.

How to measure temple temperature

- Always make sure the probe is clean, and without damage.
- Press "ON/OFF/Memory" button to power on the device.
- Vertically aim the thermometer at the temple with a distance of less than 5 cm. If the temple is covered with hair, sweat or dirt, remove the obstacle from the temple to improve the measuring accuracy.
- Press the "Start" button to take temperature. Measurement is complete when you hear a long beep sound.
- Read the recorded temperature from the LCD display.
- Clean the probe after each use to ensure an accurate reading and avoid cross contamination. (See the section of Care and Cleaning for details.)
- Remember to put in the storage case when not in use.

Switching between Fahrenheit and Celsius

- Make sure the device is off. (Or you can turn off the device by pressing and holding the "On/Off/Memory" button for 3 seconds.)
- Hold onto the "Start" button, then press and hold the "On/Off/Memory" button until "C" appears on the LCD.
- Press the "On/Off/Memory" button to select the desired scale $^{\circ}\text{C}$ or $^{\circ}\text{F}$, once you decide the scale, wait for 5 seconds and the setting will be confirmed while you will hear 2 short beep sounds.

How to measure object temperature

When you want to measure object temperature, you have to change to Object Mode.

- Make sure the device is turned off.
- Hold down the "Start" button for about 8 seconds, and the measuring site icon on the display will be switched accordingly.
- Release the "Start" button when the desired measurement mode is shown and 1 short beep sound will be heard.
- When you are at Object Mode, press the "Start" button to get the real temperature. Note:
- Applications include temperature measurements for Water, Milk, Cloth, Skin or other object.
- This mode shows the actual, unadjusted object temperatures, which is different from the body temperature.

Memory function

There are total 9 set memories for measurement records. If the reading of the thermometers is within the normal temperature range of 34°C to 42.2°C (93.2°F to 108°F), when the measurement is done, the measurement data is saved into memory.

Care and cleaning


- The probe is the most delicate part of the thermometer. Use with care when cleaning the lens to avoid damage.
- After the measurement, please use the cotton swab with the Alcohol (70% concentration) to clean the inside of the probe, including lens and metal parts.
- Allow the probe to fully dry for at least 5 minutes.
- Storage temperature range: It should be stored at room temperature between -25°C to $+55^{\circ}\text{C}$, (-13°F to $+131^{\circ}\text{F}$) RH $\leq$ 95%.
- Keep the unit dry and away from any liquids and direct sunlight.
- Please check the device if damaged once it falls. If you can't make sure of it, please send the complete device to the nearest retailer for re-calibration.
- Holding the thermometer too long may cause a higher ambient temperature reading of the probe. This could make the body temperature measurement lower than usual. Please try not to touch the thermometer probe part for avoiding the incorrect reading.

Battery Replacement

- Open the battery cover: use thumb to push battery cover out.
- Hold the device and flip the battery out.
- Insert the new battery down with the correct electrical polarity.
- Place back the battery cover.
- Keep the battery away from children.

Troubleshooting:

Error Message	Problem	Solution
EE	The system is not functioning properly.	Unload the battery, wait for 1 minute and re-power it. If the message reappears, contact the retailer for service.
Ei	Exceeding operating temperature range	Allow the thermometer to rest in a room for at least 30 minutes at room temperature: $15\text{--}40^{\circ}\text{C}$ ($59\text{--}104^{\circ}\text{F}$).
Hi	(1) In Temple mode: Temperature taken is higher than $+42.2^{\circ}\text{C}$ (108°F) (2) In Object mode: Temperature taken is higher than 99.9°C (199.9°F)	Please select the target within specifications. If a malfunction still exists, please contact the nearest retailer.
Lo	(1) In Temple mode: Temperature taken is lower than $+34^{\circ}\text{C}$ (93.2°F) (2) In Object mode: Temperature taken is lower than 0°C (32°F)	Please select the target within specifications. If a malfunction still exists, please contact the nearest retailer.
PIPED 1888	Device cannot be powered on to the ready stage.	Change with a new battery.
	Low battery: Battery icon is flashing, still can be measurable.	Suggest to replace the battery.
	Dead battery: Battery icon always enabled, no more measurements are possible.	Replace the battery

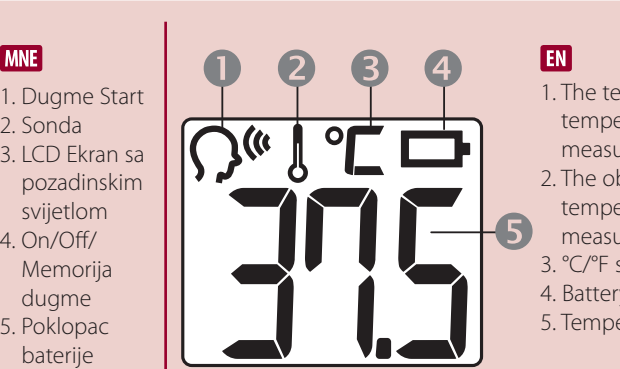Specification

Temperature measurement range	Temple mode: $34\text{--}42.2^{\circ}\text{C}$ ($93.2\text{--}108^{\circ}\text{F}$) Object mode: $0\text{--}99.9^{\circ}\text{C}$ ($32\text{--}199.9^{\circ}\text{F}$) For temple mode: $\pm 0.2^{\circ}\text{C}$ (0.5°F) during $34\text{--}42.2^{\circ}\text{C}$ ($93.2\text{--}108^{\circ}\text{F}$). For object mode: $\pm 0.2^{\circ}\text{C}$ or $\pm 0.6^{\circ}\text{F}$ when measuring the object temperature between $0\text{--}99.9^{\circ}\text{C}$ (or $32\text{--}199.9^{\circ}\text{F}$).
Accuracy	Temple mode: $15\text{--}40^{\circ}\text{C}$ ($59\text{--}104^{\circ}\text{F}$), RH $\leq$ 95%; $700\text{--}1060$ hPa Object mode: $10\text{--}40^{\circ}\text{C}$ ($50\text{--}104^{\circ}\text{F}$), RH $\leq$ 95%; $700\text{--}1060$ hPa
Operating condition	Temperature: $-25\text{--}55^{\circ}\text{C}$ ($-13\text{--}131^{\circ}\text{F}$), RH $\leq$ 95%; $700\text{--}1060$ hPa
Storage and transportation condition	9 set
Memory	0.1
Display resolution	AAA x 2
Battery	55g (without battery)
Weight (without battery)	Size
Size	14.5cm(L) x 3.95cm(W) x 4.0cm(H)
Auto shutdown	60 sec
Battery life	3000 consecutive measurements or 1 year with 1-2 measurements per day including stand-by mode.
Safety classification	IP22 Protection against water and dust
IP Classification	

* Dispose of device and batteries according to local regulations

Caution

- Only use the device once you have read and understood these instructions for use. Keep the instruction manual for use.
- The thermometer and patient needs to stay in the room in which the measurement is taken for at least 30 minutes before use.
- It is recommend that you always take the temperature in the same location, since temperature readings may vary according to the locations.
- It is recommended that three temperatures are taken and the highest one taken as the reading if:
 - New born infants
 - Children with a compromised immune system and for whom the presence or absence of fever is critical.
 - When the user is learning how to use the thermometer for the first time until he/she has familiarized himself/herself with the instrument and obtains consistent readings.
- The reading is extremely low.
- Do not take a measurement while drinking, eating, sweating or exercising.
- Do not take measurement while or immediately after nursing.
- Children is not allowed to use the device. Medical products are not toys.
- Influences on body temperature
 - A person's individual metabolism
 - Age: Greater temperature fluctuations occur faster and more often in children. Normal body temperature decreases with age.
 - Clothing
 - Outside temperature
 - Time of day: Body temperature is lower in the morning and increases throughout the day towards evening.
- Activities: Physical and, to a lesser extent, mental activities increase body temperature.

- This device is not shock-protected. Do not drop or expose to heavy shock this device.
- Do not bend the device.
- Do not disassemble or make modifications the device.
- Do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.
- Do not boil the probe.
- Do not use the device if it operates irregularly or any error message showed.
- Do not use too thinner or benzene to clean the device.
- Wipe the device clean before storing.
- When take the device from storage at below or above $15\text{--}40^{\circ}\text{C}$ ($59\text{--}104^{\circ}\text{F}$), place it in $15\text{--}40^{\circ}\text{C}$ ($59\text{--}104^{\circ}\text{F}$) temperature range for at least 30 minutes before use.
- Remove the battery if the device will not be used for a long time.
- If this device is used according to the operation instruction, periodic re-calibration is not required. If you still have questions, please send the complete device to dealers. Measurement results are for reference only. Contact your physician if you have or suspect any medical problems. Do not change your medications without the advice of your physician or healthcare professional.
- This device may not meet its performance specification if stored or used outside temperature and humidity ranges specified in specifications.
- Battery should not be charged or placed into extreme heat as it may explode.
- The thermometer contains small parts (e.g. battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Be sure to consult a doctor if you feel that your health is in poor condition.
- Do not judge your health only on the presence or absence of a fever.
- Do not service or maintain device while in use.
- This product is certified as a medical device in the European Union under the Medical Device Directive 93/42/EEC by SGS CE1639, exclusively for the indication(s) of human body temperature. Other non-medical uses ascribed to this device have not been evaluated by SGS and are not within the scope of CE certification. Users should be aware that product performance and/or safety has not been evaluated by SGS for those purposes.

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